**Articulating Pectoral Model**

**Sternal Pad Model**

### Step 1: Adjust Horizontal Bars

- Loosen the four Allen screws on the horizontal bars.
- Position the brace on the patient so that the Horizontal Assembly is located approximately 3" below the xiphoid process.
- Slide the horizontal bars out until the outside edge of each lateral pad is ½ inch away from the body. The pads will pull in next to the body when the strap is tightened.
- Remove brace from patient. Tighten the four Allen screws in the horizontal bars.

### Step 2: Adjust Vertical Bars

- Loosen the four Allen screws on the vertical bars.
- With patient either sitting or lying on back place brace on chest and...
  - Articulating Pectoral Model:** Slide vertical bars until the pectoral pads rest below the clavicle bone on soft tissue, or
  - Sternal Pad Model:** Slide vertical bar until top of the sternal pad is positioned slightly inferior to the sternal notch
- The bottom of the public pad should be three finger widths (2") superior to the pubis.
- Remove brace from patient. Tighten all the button-head screws with the Allen wrench.

### Step 3: Strap & Pad Adjustments

- On the side that the patient uses to fasten the orthosis, slip the end to the strap velcro side up through the lateral pad slot, until the strap seam aligns with the slot. Fasten strap by pressing the hook into the pile fabric.
- Slip the free end of the strap under/around the patient's back and insert it through the opposite lateral pad slot. Pull until snug.
- Mark the inner side of the pile strap where it meets the lateral pad slot. (This will serve as a guide for the patient just in case they remove this side of the strap accidentally).
- Loosen the originally fastened side (see A above). Slide the posterior pad (foam toward patient) onto the strap.
- Insert the free end of the strap back into the lateral pad slot and press the hook into the pile fabric.

### Check the Fitting & Teach the Patient To Apply

- Check for patient comfort while patient is sitting and standing:
  - The sternal pad should not be tight on the throat.
  - The pectoral pads should rest below the clavicle bone on soft tissue.

### Cautions With the C.A.S.H. Orthosis

- DO NOT for any reason loosen the screws and re-adjust the brace! If it feels too long when you sit down, shift it downward to relieve neck discomfort.
- Keep brace away from excessive heat (radiators, oven tops, etc.). The C.A.S.H. Orthosis will withstand body-heat or direct sunlight, but may lose its shape if in prolonged contact with temperatures greater than 210° F.
- Cover the hooks or end parts of the strap. If exposed, it can snag clothing.

### QUESTIONS

Contact Customer Service at:

USA

Tel: +1 800 492 1088

Fax: +1 800 245 3765

Email: [info-usa@trulife.com](mailto:info-usa@trulife.com)

Visit Trulife online at [www.trulife.com](http://www.trulife.com).

Canada

Tel: +1 800 267 2812

Fax: +1 613 392 4139

Email: [infocanada@trulife.com](mailto:infocanada@trulife.com)

Trulife has appointed Medical Device Safety Service (MDSS) of Hannover, Germany to act as our EU authorized representative. They may be contacted at:

**MDSS GmbH**

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30175 Hannover

Germany

**Phone** (+49)-511-6262 8630

**FAX** (+49) -511-6262 8633

### LIMITED WARRANTY

Trulife warrants that the PRODUCT will be free from defects in material and workmanship for the PRODUCT from the date of installation.

This warranty will not apply if the PRODUCT has been damaged by misuse, abuse, neglect, improper care, failure to follow instructions, abnormal wear and tear, or in the event that a PRODUCT has been modified/repared by persons unauthorized by Trulife.

If a defect in material or workmanship is found during the warranty period, Trulife will, at Trulife's option, either repair or replace the product. If it is not possible to repair or replace the product, Trulife will be limited to refunding the purchase price.

Trulife will not be liable under any legal theory for any direct, indirect, special, incidental or consequential damages arising from the use of or inability to use this product.

The application guidelines for this Trulife product are for the use of and by certified, qualified practitioner only. Patients are not to attempt to apply or adjust the item unless expressly instructed to do so by the practitioner responsible for the prescription and/or initial fitting of the device. All patient questions should be referred to the practitioner and not to the manufacturer. The manufacturer warrants only that the enclosed product has been inspected for quality and can be effective for certain indications, but final decisions and ongoing monitoring must be made by the medical professional(s) prescribing and/or fitting the device to determine its effectiveness for an individual patient. Patient compliance is an integral part of the entire protocol and must be adhered to in order to avoid potential problems and to maximize the effectiveness of the prescribed product.

As a condition of the sale of any Trulife product, this product is restricted to a "Single Patient Use Only" by the originally fitted patient in order to protect the care provider and the patient against potentially adverse consequences of infectious disease transmission, material instability in adapting to the configuration of the original user and/or decrease in effectivity. Any express or implied warranties are voided if the product is reused or fitted to another patient. Additionally, a license of right to use under any relevant patents pertaining to the product is terminated with the cessation of use by the original patient. As with all Trulife products, this product must be prescribed and applied by a qualified practitioner to determine it meets the needs of the particular patient and accomplishes the desired results.

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