



INTRODUCTION

The Green Dot Valves are designed to provide vacuum suspension through manual expulsion. Each Green Dot valve and housing set is furnished with a disposable valve plug, a screw and a washer for fabrication purposes, and a valve key for patient use.

Product Code	Description
P123101000	Green Dot Total Contact
P123401000	Green Dot SFS

INSTALLATION AND USE

1. Screw the plug into the valve housing.
2. Drill a 3.2 mm ($\frac{1}{8}$ ") pilot hole at the center of the determined valve location.
3. Thread a nylon stocking through the hole in the plug and tie a knot(s) to prevent the plug from slipping off.
4. Pull the nylon over the model, positioning the plug over the pilot hole and by temporarily displacing the knot, securely attach the plug and housing to the model with the screw and washer provided.
Note: If nylon is not used, an air channel is necessary through the cast to provide vacuum at the valve site.
5. Vacuum form the plastic. Light constriction around the housing in the soft plastic with a cord will help the vacuum pull the plastic snugly.
6. After the plastic is cool, the excess plastic over the plug should be removed with a sharp utility knife and the plug removed and discarded.

Sweat or debris such as powder can become lodged in the valve over time and may cause a decrease in vacuum performance. Remove the valve from the housing and clean the valve using warm water and a mild soap. Rinse the valve thoroughly and towel dry.

Warning: Failure to follow the installation and use procedures set forth above may lead to structural failure of the components subjecting the user to a risk of serious personal injury.

QUESTIONS

Contact Customer Service in the U.S. at 888-878-1238, or fax 888-878-1237

Contact Customer Service in Canada at 800-267-2812, fax 613-392-4139

If calling from outside the U.S. or Canada, contact Customer Service at 360-697-5656, or fax 360-697-6843 Visit Trulife online at www.trulife.com

LIMITED WARRANTY

Trulife warrants that the Green Dot Valves will be free from defects in material and workmanship for two (2) years from the date of installation.

This warranty will not apply if the product has been damaged by misuse, abuse, neglect, improper care, failure to follow instructions, abnormal wear and tear, or in the event that the Green Dot Valve has been modified/repaired by persons unauthorized by Trulife.





If a defect in material or workmanship is found during the warranty period, Trulife will, at Trulife's option, either repair or replace the product. If it is not possible to repair or replace the product, Trulife will be limited to refunding the purchase price.

Trulife will not be liable under any legal theory for any direct, indirect, special, incidental or consequential damages arising from the use of or inability to use this product.

The application guidelines for this Trulife product are for the use of and by certified, qualified practitioner only. Patients are not to attempt to apply or adjust the item unless expressly instructed to do so by the practitioner responsible for the prescription and/or initial fitting of the device. All patient questions should be referred to the practitioner and not to the manufacturer. The manufacturer warrants only that the enclosed product has been inspected for quality and can be effective for certain indications, but final decisions and ongoing monitoring must be made by the prosthetic professional(s) prescribing and/or fitting the device to determine its effectiveness for an individual patient. Patient compliance is an integral part of the entire protocol and must be adhered to in order to avoid potential problems and to maximize the effectiveness of the prescribed product.

As a condition of the sale of any Trulife product, this prosthesis is restricted to a "Single Patient Use Only" by the originally fitted patient in order to protect the care provider and the patient against potentially adverse consequences of infectious disease transmission, material instability in adapting to the configuration of the original user and/or decrease in effectivity. Any express or implied warranties are voided if the product is reused or fitted to another patient. Additionally, a license of right to use under any relevant patents pertaining to the product is terminated with the cessation of use by the original patient. As with all Trulife products, this product must be prescribed and applied by a qualified practitioner to determine it meets the needs of the particular patient and accomplishes the desired results.

