INTRODUCTION AND IMPORTANT NOTE

Trulife’s Precision Pro Knee Brace is designed to provide support and protection for the knee. Because of variations in each patient’s physical attributes, medical condition and therapeutic limitations, the Precision Pro Knee Brace is prescribed by a physician. The level of physical activity and accompanying stress to the knee joint that a patient can tolerate should be something discussed carefully with your physician.

Contact sports and high-velocity sports are inherently dangerous and may result in serious injury. Although these knee braces may help reduce the risk of injury or may diminish the extent of any injury to the knee joint, use of the Precision Pro is not guaranteed to prevent any such injuries. No brace can completely prevent injuries to the knee joint, and as such, no guarantee is hereby made with regard to the Precision Pro Knee Brace.

INSTALLATION AND USE

Recommended installation and use procedures must be followed for maximum safety and service life.

Warning: Do not modify the frame of knee brace except to shape it as instructed in this guide. It will void the warranty and can cause product failure.

1. Locate knee center by palpating the superior angle of the medial tibial plateau. “Knee Center” is 1.9 cm (0.75”) above this point (FIGURE 1).
2. With the patient seated and the affected leg flexed at between 10° and 15°, apply the brace to the knee area.
3. Align the condyle pads with knee center.
4. For ACL instabilities (FIGURE 2):
 - A. Push the proximal tibia back by firmly tightening the strap over the tibial tubercle.
 - B. Pull the distal femur forward by tightening the strap above the popliteal area.
 - C. Firmly secure the remaining straps (above the patella and below the popliteal area).
5. For PCL instabilities (FIGURE 3):
 - A. Pull the proximal tibia forward by firmly tightening the strap over the calf.
 - B. Push the distal femur back by tightening the strap over the patella.
 - C. Firmly tighten the remaining straps (above the popliteal area and below the patella).
6. Provide patient with Patient Instructions for Precision Pro Knee Brace.
7. Instruct the patient to undo only the straps at the back of the leg when removing the brace.
8. Set range of motion. Range of motion is preset at 10° on the brace. An extension/flexion stop kit is included for field ROM adjustments.

Warning: Failure to follow the installation and use procedures set forth above may lead to structural failure of the components subjecting the user to a risk of serious personal injury.

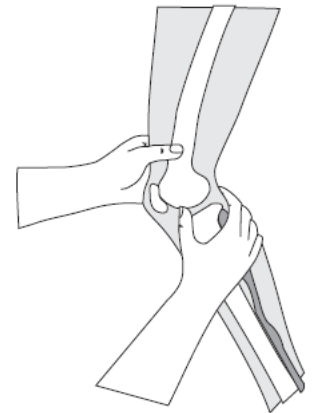
Figure 1: Find Knee Center.

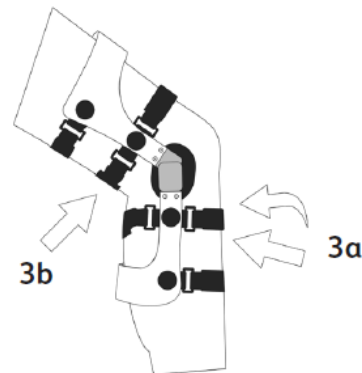
Figure 2: For ACL instabilities.

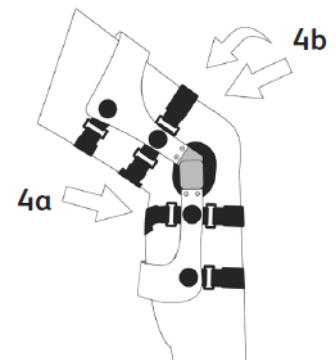
Figure 3: For PCL instabilities.

QUESTIONS

Contact Customer Service at;

USA

Tel: +1 800 492 1088

Fax: +1 800 245 3765

Email: info-usa@trulife.com

Visit Trulife online at www.trulife.com.

Canada

Tel: +1 800 267 2812

Fax: +1 613 392 4139

Email: infocanada@trulife.com

Trulife has appointed Medical Device Safety Service (MDSS) of Hannover, Germany to act as our EU authorized representative. They may be contacted at:

MDSS GmbH

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30175 Hannover

Germany

Phone (+49)-511-6262 8630

FAX (+49) -511-6262 8633

LIMITED WARRANTY

Trulife warrants that the PRODUCT will be free from defects in material and workmanship for the PRODUCT from the date of installation.

This warranty will not apply if the PRODUCT has been damaged by misuse, abuse, neglect, improper care, failure to follow instructions, abnormal wear and tear, or in the event that a PRODUCT has been modified/repared by persons unauthorized by Trulife.

If a defect in material or workmanship is found during the warranty period, Trulife will, at Trulife's option, either repair or replace the product. If it is not possible to repair or replace the product, Trulife will be limited to refunding the purchase price.

Trulife will not be liable under any legal theory for any direct, indirect, special, incidental or consequential damages arising from the use of or inability to use this product.

The application guidelines for this Trulife product are for the use of and by certified, qualified practitioner only. Patients are not to attempt to apply or adjust the item unless expressly instructed to do so by the practitioner responsible for the prescription and/or initial fitting of the device. All patient questions should be referred to the practitioner and not to the manufacturer. The manufacturer warrants only that the enclosed product has been inspected for quality and can be effective for certain indications, but final decisions and ongoing monitoring must be made by the medical professional(s) prescribing and/or fitting the device to determine its effectiveness for an individual patient. Patient compliance is an integral part of the entire protocol and must be adhered to in order to avoid potential problems and to maximize the effectiveness of the prescribed product.

As a condition of the sale of any Trulife product, this product is restricted to a "Single Patient Use Only" by the originally fitted patient in order to protect the care provider and the patient against potentially adverse consequences of infectious disease transmission, material instability in adapting to the configuration of the original user and/or decrease in effectivity. Any express or implied warranties are voided if the product is reused or fitted to another patient. Additionally, a license of right to use under any relevant patents pertaining to the product is terminated with the cessation of use by the original patient. As with all Trulife products, this product must be prescribed and applied by a qualified practitioner to determine it meets the needs of the particular patient and accomplishes the desired results.

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