



Trulife #780 Fast-Wrap Thoracolumbar Support


Instructions for use / Instructions d'usage



Center the back



Attach the front

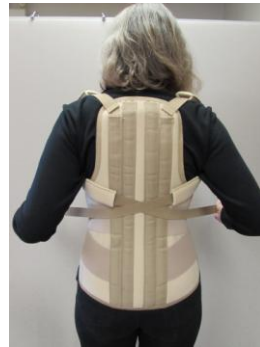

Cross the shoulder straps



Attach straps to the front.



Shape and insert the steel stays



Reapply. Fitted view, front.



Fitted view, back.

Instructions

- Spinal braces should be worn over a close fitted shirt or undershirt
- Prior to laundering, take out all the removable stays/hinges/panels and close all hook and loop straps onto themselves to prevent exposed hook material from damaging other materials.
- Do not use fabric softener

Instrucciones

- Tirantes Espinales deben ser usados sobre una camisa estrecha o camiseta interior.
- Antes de lavar, tome todas las estancias extraíble/bisagras/paneles y cierre todas las correas gancho y bucle para evitar el daño de los ganchitos expuestos a otros materiales.
- No use suavizante.

Instructions

- Les supports dorsaux doivent être portés par-dessus un tee-shirt bien ajusté.
- Avant le lessivage, enlever toutes les articulations, baleines et panneaux et attacher les courroies crochet et velours sur elles-mêmes pour empêcher d'abîmer tout autre matériel.
- Ne pas utiliser d'assouplissant textile.

Trulife USA and International Tel: 1.360.697.5656, Fax: 1.360.697.6843

Trulife Canada Tel: 1.800.267.2812, Fax: 1.613.392.4139

www.trulife.com